

PREVENTOGEN™

INFECTION PREVENTION POLYMER

Instructions for Use

FOR EXTERNAL USE ONLY

20 ML BOTTLE

INGREDIENTS

Organic polymer in methylene chloride organic solvent

INTENDED USE AND PRODUCT DESCRIPTION

- Preventogen is indicated for providing a protective covering over closed surgical incisions and excisions, minor wounds and scrapes that are clean and dry
- Forms a microbicidal barrier [destructive to microbes]
- Preventogen provides an organic, clear, elastomeric, flexible, water resistant, non-odorous film for covering minor wounds, cuts, abrasions, burns and scrapes
- The film protects the wound against water, dirt and germs and is not intended as a treatment or cure for all types of wounds
- The elastomeric properties of the film help protect in difficult body regions where flexing, bending and creasing skin occurs
- Protects wound during healing
- Protects skin from blisters, calluses, and sores due to rubbing

PREPARATION

- Before applying Preventogen remove any moisturizers, lotions, soaps or creams thoroughly, clean and dry the skin
- Protect sensitive areas, such as stomas and mucous membranes (mouth, eyes and inner nose)

APPLICATION

- Apply the liquid evenly in a counter clockwise orientation around the wound
- Cover an area that extends beyond the edges of the wound at least 1.25 in
- Momentary stinging may occur
- When used at room temperature, Preventogen will dry in less than one minute and form and polymerize into an elastomeric barrier
- Inspect film to be sure a complete, continuous covering of the wound and surrounding area has occurred

APPLICATION CONT

- Preventogen will cover an area approximately 5 in x 5 in, or 2 in x 11 in
- Ensure that Preventogen has completely dried before covering with clothing, additional dressings, adhesives or athletic tape
- Depending on skin type, humidity, activity level and friction on the application surface, Preventogen will naturally wear off over 24 to 72 hours, but may last longer when covered with additional dressing and/or areas that receive less friction and use
- You may begin to see raised edges of the Preventogen polymer after swimming, showering or physical activity
- The dry film covering should be inspected frequently for leakage and loosening of the edges
- Additional layers may be applied if the dry film begins to slough off or peel around the edges
- Following application, the bottle should be tightly capped to prevent evaporation

REMOVAL

- To remove the film, wash the area with soap and water and gently peel off starting at the outer edges

STORAGE

- Store at room temperature between 68-77°F
- Do not expose product to direct sunlight

CAUTION

- For external skin use only.
- Do not use on deep or infected wounds or puncture wounds. Contact a physician for deep, puncture or infected wounds, animal bites, redness, swelling, or pain
- Intentional inhalation of the contents may be harmful or fatal. Use with adequate ventilation. Avoid contact of the solution with clothes or finished surfaces. Do not store near heat or flames.
- Do not place in plastic basin.
- Bottle should be tightly capped after use to prevent evaporation of the solvent.
- Keep away from children, do not use near eyes, mouth, or nose. In case of ingestion, obtain medical assistance immediately

CONTENTS

- 0.6 fl oz solution contained in 20 mL glass bottle